

A close-up portrait of a woman with dark hair pulled back, looking slightly to the left. Her skin is exceptionally smooth and has a natural-looking glow, particularly on her forehead and cheeks. The lighting is soft and directional, highlighting the texture of her skin. The background is a plain, light-colored wall.

REGENERATE | REVIVE  
**GLOREV**  
Made in Spain

**ADVANCED REJUVENATION FOR TIMELESS SKIN**

# GLOREV- A REVOLUTIONARY BIO-REVITALIZING COMPLEX DESIGNED TO TIGHTEN, HYDRATE, RESTORE & ILLUMINATE THE SKIN FROM WITHIN



**THE PERFECT BLEND FOR  
10 YEARS YOUNGER SKIN**



**NON-CROSSLINKED HYALURONIC  
ACID FOR YOUTHFUL RENEWAL  
THROUGH MESOTHERAPY**

# Hydrate. Lift. Tighten. Brighten.

## The Power of Four Actives in One Formula

### Hyaluronic acid (low weight) 1%

- Deep dermal penetration to lift & tighten the sagging skin
- Activates fibroblasts to produce more collagen & elastin
- Refines skin texture & elasticity
- Smooths wrinkles & fine lines
- Revitalizes & repairs skin at cellular level

### Hyaluronic acid (medium weight) 1%

- Resides in the mid-dermal matrix, facilitating prolonged water retention.
- Intense & long lasting hydration
- Improves texture, firmness & elasticity
- Restores skin balance & resilience
- Restores under-eye volume for a refreshed, youthful look

### Glutathione 1%

- Potent antioxidant
- Reduces melanin synthesis, leading to a lighter, even-tone skin
- Fades dark spots, hyperpigmentation & melasma
- Revitalizes dull, aging skin for a healthier, luminous complexion
- Brightens dark under eyes for a refreshed look

### PDRN (Salmon extract) 0.3%

- Stimulates tissue regeneration to boost collagen synthesis
- Heals scars & damaged skin
- Stimulates dermal remodeling & strengthens structure
- PDRN boosts ECM repair, prolonging HA's hydrating effect



Sagging Skin



Lifted &  
Tightened Skin



Dry &  
Dehydrated Skin



Plumped &  
Hydrated Skin



Dark Dull Skin



Brightened &  
Even Toned Skin



Damaged  
Scarred Skin



Restored &  
Regenerated Skin

## HOW TO USE:

- Numb and cleanse the treatment area.
- Use micro needling / Nappage technique (inject 0.05–0.1 ml/point, spaced 0.5–1 cm, at 0.2–0.6 mm depth) (For dull, dehydrated & uneven skin tone)
- For under eyes use Cleopatra mask technique (0.02–0.05 ml/point; depth: 0.2–0.3 mm) (For crow's feet & drooping eyes)
- Use the BAP technique. (Five points per side, 0.2 ml/point) (For sagging skin, fine lines & wrinkles)
- Avoid makeup & sun for 24–48 hours; use SPF 50 PA++ outdoors.

## RECOMMENDED PROTOCOL

**Initial Phase:** 2 sessions 4-6 weeks apart or as needed depending on the skin's condition, age and lifestyle.

**Maintenance Phase:** 1 session every 6 months to 1 year

## VOLUME PROTOCOL

**FOREHEAD**  
(0.5 - 1.5 ml)

**UNDEREYES**  
(0.5 - 1 ml)

**FACE**  
(0.5 - 2 ml)

**NECK**  
(1.5 - 2 ml)

**BACK OF HANDS**  
(1 - 2 ml)


**DÉCOLLETAGE**  
(0.5 - 1.5 ml)



## BAP TECHNIQUE

(For safe, even diffusion & deep bioremodeling)

Five points per side  
(Total 10 Points)  
Massage: Not required  
(product diffuses naturally)

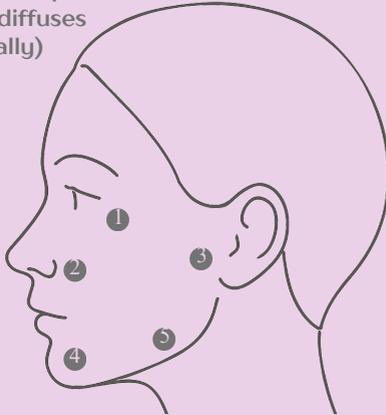


- 1 Zygomatic Protrusion (malar area)
- 2 Nasal Base
- 3 Pre-Auricular Area (before tragus)
- 4 Chin / labial commissure
- 5 Mandibular Angle

Downtime: Minimal; slight swelling may resolve within 24–48 hours

## NAPPAGE TECHNIQUE

(For superficial hydration & glow)


Depth: 0.2–0.6 mm  
Spacing: 0.5 – 1 cm  
Angle: 30°– 60°  
Massage: light (optional)

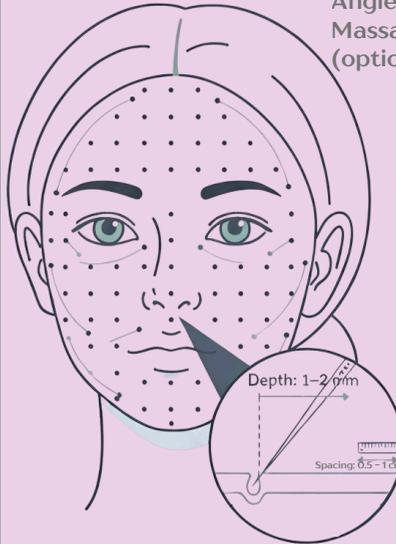

Downtime: Minimal; mild redness may subside within 24 hours

## CLEOPATRA MASK TECHNIQUE

(For periorbital area)

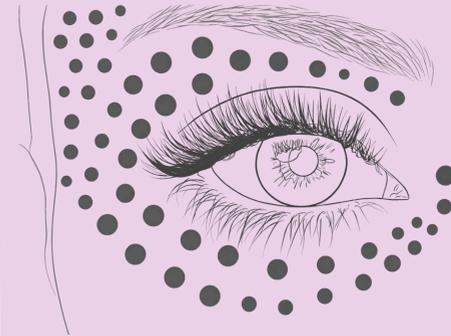


For under-eye, crow's feet, lower eyelid)  
Needle: 33G–34G, 1.5mm/2.5mm  
Depth: 1.5–2 mm


Note: Do not inject into the movable upper eyelid

## HYALURONIC ACID: DUAL ACTION SKIN REJUVENATION

### DEEP HYDRATION, FIRMNESS LIFTED & TIGHTENED SKIN

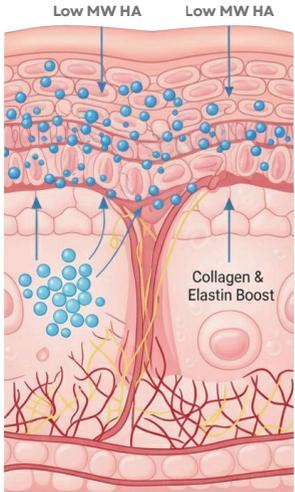

### SURFACE HYDRATION & BARRIER SUPPORT

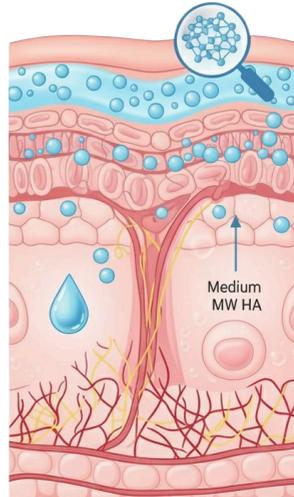

## WE RECOMMEND THIS FOR:



Those seeking to improve  
fine lines, wrinkles &  
sagging skin



Those troubled by dull  
& tired looking skin



Those seeking to improve  
skin tone, texture &  
overall luminosity



Those seeking to lift  
drooping eyes & restore  
under-eye volume



Ideal for early aging  
& sun-damaged skin.



Those aiming to improve thin,  
crepey skin on neck & hands.



For plumped  
smooth taut skin




# GLOREV- BEYOND HYDRATION, TRUE SKIN RENEWAL

Unlike a traditional filler Glorev remodels the tissue and improves skin quality without adding bulk or altering facial structure



**TURN BACK TIME —  
REDEFINE YOUR GLOW**

# NON CROSSLINKED HYALURONIC ACID | SKIN REJUVENATION & FIRMING

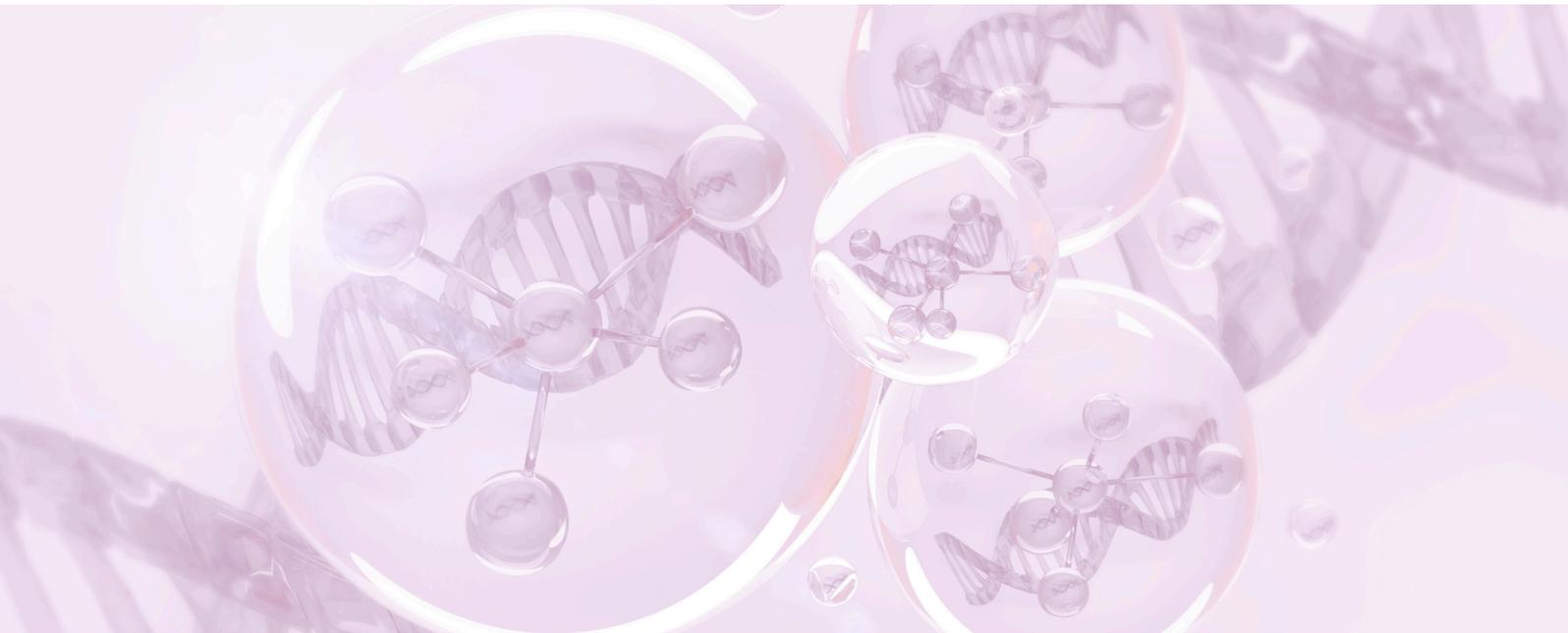

**STERILE ILLUMINATING & HYDRATING SOLUTION**

## GLOREV

Aesthetic sterile solution

Store between 15- 30°

e 1 Vial × 5 mL (0.17 fl. oz.)

Protect from sunlight

**MADE IN SPAIN**

Stable & biocompatible

Expiry: 3 years from production

[WWW.MESOREV.COM](http://WWW.MESOREV.COM)